

Quick Guide for REVO Steam and Vacuum for Wheelchair Cleaning




Turn the machine on by pressing this button.


When the machine beeps, fill the water tank here.

When the hose is connected to the hose socket, press this button to prepare the vacuum.

Wait 4-7 Minutes for the flashing light on the power button to become solid. Now you are ready to clean. The light may start to blink again while you are working. If you have steam coming out of the nozzle, the blinking is fine. You do not need to stop working and wait for it to become solid again. You can adjust the pressure of the machine with the small silver knob.

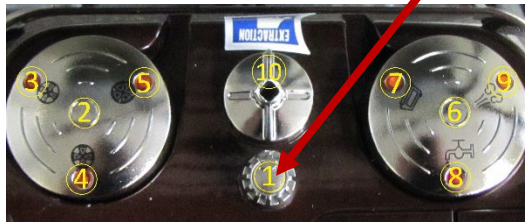

To turn on vacuum suction, press the fan symbol on the hose handle. To use steam, pull the trigger.

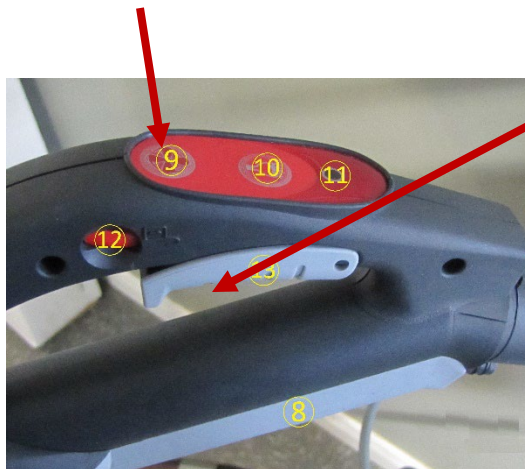

Use the smallest attachment first – you can use the nozzle – with or without the vacuum adaptor to clean crevices, corners, velcro, seat belts – anywhere the larger tools won't reach. Do this first, so any dirt you loosen from these areas will be collected when you clean the larger spots. You can use a brush if the area requires some extra scrubbing.



Once you have cleaned the detail areas, you can proceed to use the large extraction tool to steam and vacuum the large seats, backrests, foot rests and so on.

You can use this accessory on it's own, or with the adaptors for scrubbing or squeegee.



If you prefer, you can use one or both extension tubes to make reaching areas more ergonomic.



Always use care when switching accessories – they may become hot during use. Never put any chemicals in the water tank.

When you are finished, unplug the machine and return it to the storage area. If anything is wrong with the machine, let someone know right away and it will be fixed.